



# Using your Thermal Printer

A quick guide to operation

274408D

 **Prestige Medical**<sup>®</sup>  
Integrated decontamination solutions

## A quick guide to operation

(for more detailed instructions and specification about the printer, see enclosed manufacturer's leaflet)

Ensure that there is a paper roll in the printer.

### Bench Top Autoclaves:

Connect the grey printer lead into the printer and the multi-pin connector into the autoclave printer port. (When looking from the front, this will be on the rear left hand side of the unit). Connect the black power adaptor to the printer and plug into a suitable 13 amp socket. When the indicator light comes on, the printer is ready for use.



### Portable Autoclaves:

Connect the grey printer lead into the printer and the multi-pin connector into the autoclave printer port. Connect the black power adaptor to the printer and plug into a suitable 13 amp socket. When the indicator light comes on, the printer is ready for use.

### Replacing Paper Roll:

To replace the paper roll simply open the lid by pulling the clear plastic lever forward and upwards. Remove the spool of the old paper. Reel off a few centimetres of paper from the new roll and keep approximately 5 cm outside the printer. Place the new roll into position with the paper feeding from the bottom front and with the leading edge over the serrated tear bar. Gently close the lid until it locks into position.

To re-order printer rolls (pack of 10 rolls)

Call 01254 844 103

and quote part number 279505.

NB: No batteries required. UK only.