



3M™ 8825 / 8835 Premium Respirators

Fitting Instructions

3M 8825/8835 Premium Respirators

Respirator	Type	APF
8825	FFP2 R D	10
8835	FFP3 R D	20

Standard: EN149:2001 + A1:2009

This respirator should not be used in oxygen deficient environments, for escape purposes or by people with beards/ facial hair or in atmospheres where concentrations of contaminant may exceed that deemed Immediately Dangerous to Life and Health (IDLH).



3M Health & Safety Helpline

Monday - Friday 9 to 5


0870 60 800 60 UK
1 800 320 500 IRELAND



1



2

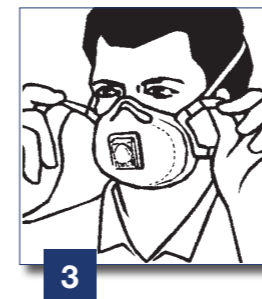


3



4



5

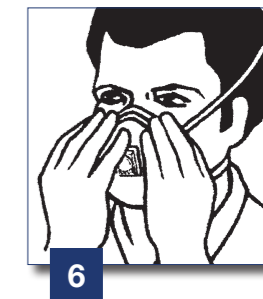

6

1. Thread top elastic strap through top buckles. Repeat for bottom strap and buckles. Place the bottom elastic strap around the head just below the ears. Untwist the strap.
2. Place the top elastic strap around the head, above the ears. Untwist the strap.
3. Adjust tension by pulling tabs of each strap, as shown.
4. Using both hands, mould metal nosepiece comfortably to shape of nose. Pinching the nosepiece using one hand may result in less effective respirator performance.
5. Strap tension may be decreased without removing respirator from the head by pushing out on the back of the buckles.
6. The seal of respirator on the face should be fit checked prior to wearing in the work area.

Fit Check Procedure

- a) Cover the front of the respirator with both hands, being careful not to disturb the position of respirator.
- b) Inhale sharply. A negative pressure should be felt inside respirator. If any air leaks around the nose, readjust the nosepiece as described in step 4. If air leaks at the respirator edges adjust the position of the respirator and/or tension of the straps. Retest the seal. Repeat the procedure until the respirator is sealed properly.

If you cannot achieve a proper fit do not enter contaminated area. See your supervisor.

Note: Do not use with beards or other facial hair that may inhibit contact between the face and the edge of the respirator.

Occupational Health & Environmental Safety Group

3M United Kingdom plc
3M Centre,
Cain Road, Bracknell
Berkshire RG12 8HT
Tel: 0870 60 800 60
www.3M.co.uk/ohes

3M Ireland Limited
The Iveagh Building
The Park
Carrickmines
Dublin 18
Tel: 1 800 320 500

Please recycle. Printed in UK.
© 3M 2009. All rights reserved.

Fit Testing: The 3M 8825/8835 Respirator is a tight fitting facepiece and therefore wearers require a fit test as per the COSHH regulations 2002. For qualitative testing - 3M™ FT10 and FT30 Fit Test Kits are available.

