


1. Align the dots if available. Gently insert all of the ampoule lid into the SnapIT. The ampoule lid becomes locked in at the neck.



2. With little effort, leverage away from the dot to snap off the lid. The lid should remain held in the SnapIT.

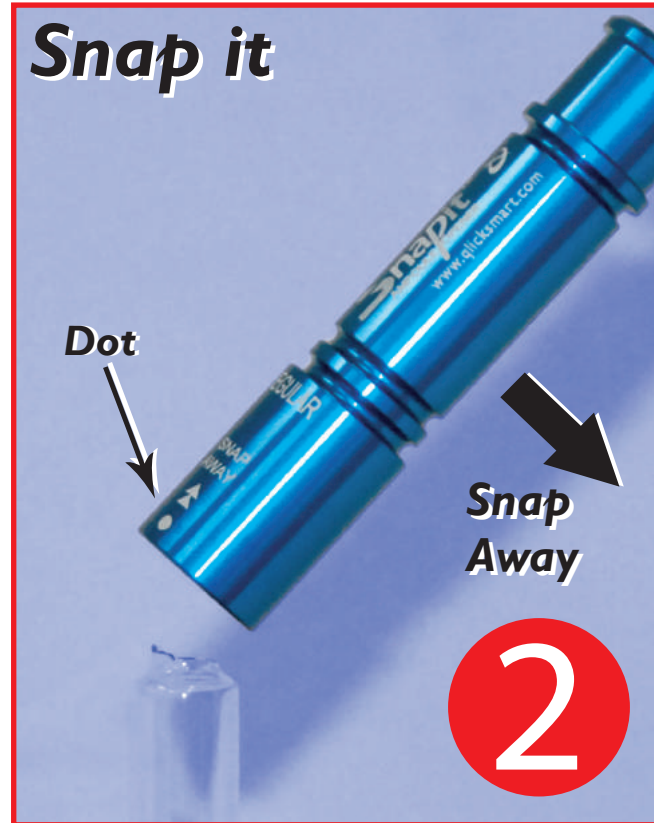

3. Carefully aim over a sharps bin and eject the lid by pushing down the metal rod with your thumb.

**NOTE:** If your standard practice requires you to turn the dots away from your body, make sure that both dots are still aligned after turning and before snapping.

**NOTE:** The lid is ejected into the sharps bin.

## Clean it



4. **Clean your SnapIT regularly.** Remove the O ring as shown to disassemble the parts and clean with water or a hospital grade cleaner. Do this in a clean dish to avoid loss of parts. Dry completely then re-assemble.

**NOTE:** If the ampoule lid breaks, tap the SnapIT over a sharps bin to remove excess glass pieces then rinse under running water before disassembling.

SnapIT Size \ Ampoule Size	REGULAR	LARGE
1-2ml	✓	-
5-10ml	✓	✓
10-15ml	✓	✓
20-25ml	-	✓

Due to the inconsistent variation of ampoule lid geometry, ampoule size in ml should be used as a guide only.

### SnapIT size is determined by ampoule lid.

- If the ampoule lid is too big, it will not fit through the SnapIT entrance. Use a larger SnapIT.
- If the ampoule is too small, the SnapIT will not hold the top securely. Use a smaller SnapIT.



### Disclaimer

It is important to read these instructions before using SnapIT. Incorrect use may not provide intended safety. It may be necessary to clean your SnapIT after opening some ampoules. Goods are warranted for 1 year against faulty workmanship. Physical damage or loss of parts will void all warranties. Receipt must be presented for all warranty claims.