

# 3 Things to Consider When Choosing your Surface Wipes

azo™

Do I need  
to clean  
the surface?

## Detergent wipes

**Detergents**, whose main purpose is to **release dirt from a surface**, are essential to the cleaning process.

**Azo™ Detergent** wipes offer a convenient and effective alternative to traditional cleaning, specifically designed to remove dirt and organic matter from surfaces. Detergent wipes have been shown to physically **remove over 99.9% of surface contamination** without the need for any harsh chemicals or disinfectants.

Did you  
know?



Flushable and  
Maceratable  
available!

Do I need  
to disinfect  
the surface?

## Alcohol wipes

**Alcohol** is a widely-used disinfectant, effective against pathogens which can cause infection.

**Azo™ 70% IPA** wipes offer a **powerful quick-drying disinfectant** with fast kill rates, effective against MRSA, E.coli and a range of other pathogens including **Coronavirus**.

Did you  
know?



CE marked options  
are also available.  
These are red!

Do I need  
to do both?

## Universal wipes

**Two-in-one surface cleaning and disinfection wipes** are convenient as they are a combination of detergent and disinfectant.

**Azo™ Universal** combines proven effective biocides with a powerful detergent to **effectively clean and disinfect surfaces**, offering broad spectrum efficacy against a range of bacteria, yeasts, fungi and viruses including **Norovirus** and **Coronavirus**.

Did you  
know?



Available in a  
variety of formats,  
including smaller  
flow-wrap packs.

# How To Use Each Product



Good cleaning and disinfection practices will help ensure the risk of cross-contamination is reduced and that maximum product efficacy levels are achieved. See below guide to using Azo™ wipes and sprays.

1.



Observe and select correct PPE and product for specific requirement.

2. If using an Azo™ wipe, remove wipe from packaging and ensure the pack is resealed to avoid wipes drying out.



3.



If using an Azo™ spray, apply directly to the surface and wipe with a clean cloth.

4.



Wipe over the surface working from clean to dirty. Do not go over the same area twice with the same wipe.

5. Ensure surface remains wet for target contact time.



6.



Once used, discard appropriately. Only clearly labelled wipes can be disposed of in a macerator or flushed down the toilet.

## Surface Compatibility

Azo™ wipes and sprays are compatible with most non-porous surfaces and equipment found in healthcare, scientific and hospitality sectors.

For sensitive equipment or uncommon surfaces, we would recommend contacting the manufacturer of these materials for guidance.

