

## Consumables for the Shandon Cytospin® Cyto centrifuges

ALL PLASTIC Single and Double CytoSep™ Cytology Funnels



These are so easy to use, and improve turn-around time while giving more diagnostic information. Being totally disposable, they eliminate time-consuming decontamination and cleaning. They are safe to use and reduce the risk of exposure to pathogenic samples. They also decrease the possibility of cross-contamination.

This series of Simport® Single CytoSep™ Cytology Funnels is single-use. The disposable sample chambers do not need metal clips and are designed to meet any processing requirement. They are capable of producing high quality thin-layer slide preparations while improving laboratory efficiency.

The disposable Sample Chambers with White Filter Cards are used for sample volumes up to 0.5 ml; the ones with the Brown Filter Cards are for sample volumes up to 0.4 ml. The cell deposition area is 6 mm in diameter (28 mm squared). Filter cards are pre-attached. Excellent for scanty specimens such as CSF. All components also available separately.

This CytoSep™ Cytology Funnel allows two sample deposition areas on a single slide. The disposable sample chambers do not need metal clips. inserted filter card. This Sample Chamber is also disposable.

Cat. #	Description	Qty/Pk	Qty/Cs
M965-10FW	Single Funnel with White Filter & Cap	40	480
M965-10FT	Single Funnel with Tan Filter & Cap	40	480
M965-20FW	Double Funnel with White Filter & Cap	40	480

For IVD use



**YOU CAN ALSO REUSE OR ACQUIRE ANY INDIVIDUAL COMPONENTS WHEN NEEDED**



Each component is strong enough to be reused.

Cat. #	Description	Qty/Pk	Qty/Cs
M965-1	Single Funnel only	50	500
M965-1D	Double Funnel only	50	500
M965B	Base only	50	500
M965C	Cap only	50	500
M965FW	White Filter Paper for Single Funnel	200	–
M965FT	Tan Filter Paper for Single Funnel	200	–
M965FWD	White Filter Paper for Double Funnel	200	–

For IVD use



Insert funnel into base.



Insert microscope slide.



Place new filter onto microscope slide.



Clip top of funnel and base together.



Your CytoSep ALL PLASTIC Cytology Funnel is now ready to use.